


Effective testing with Ecotest COVID-19 self-test

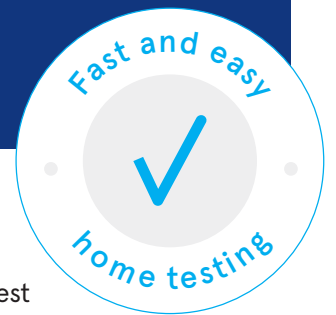


Fast and ready to use

The Ecotest COVID-19 antigen self-test is easy to take along and you can perform the test wherever you are. It is rapid and requires only a few steps to the test result. The test is single packed, and the package contains everything needed to perform the test.


Easy to use

The broader swab reduces unpleasant sensation while taking the sample and you catch more of the viral load using it. When the test device pen is inserted into the cap after sampling, the buffer compartment is sealed off. The closed system reduces the contamination risk. The test is also safe to dispose of as the buffer contains virus inactivating substances.


Only 3 steps!

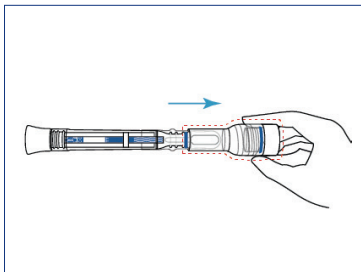

1. Take the sample collector pen out of the cap and remove the protector from the swab.

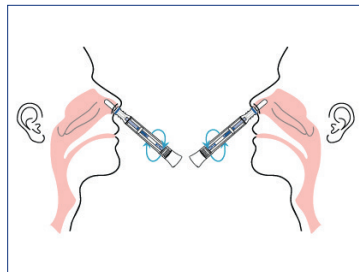

2. Insert the sample collector swab into the nostril and rotate swab five times against the nasal wall. Repeat for the other nostril.

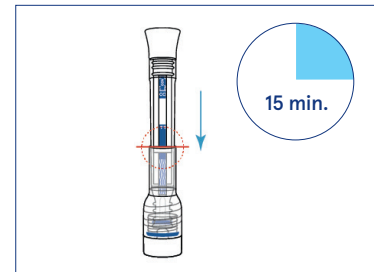

3. Place the sample collector pen back into the cap. Read the results at **15 minutes**.

Children aged between 2 and 18 years old, must be supervised or aided by an adult when carrying out the test. Test is not suitable for children under 2 years of age.
Rapid self-test should not be used as substitute for a coronavirus test performed in primary health care and it does not exempt from isolation or quarantine prescribed by a physician.

Aidian Oy

aidian@aidian.eu / +358 10 309 3000

orders@aidian.eu

www.aidian.eu

Ecotest COVID-19 Ag Pen Self-test Nasal

Package

1 test

5 tests

Ordering no

155083

155076

Watch [a video](#) on performing the test.