

QuikRead go[®] Instrument software 8.1.1 release note

A new QuikRead go Instrument software version 8.1.1 will be released in March 2020. With the new software version it will be possible to measure the new QuikRead go[®] HbA1c test.

The new software version has been developed and validated in compliance with IEC 62304:2006+AMD1:2015 Medical device software - Software life cycle processes and General Principles of Software Validation; Final Guidance for Industry and FDA Staff, Jan 11 2002.

Testing included measurement of 42 samples. All results passed.

An instrument with any of the previous software versions can be updated to this new software version. Version update does not delete old results from the memory of the instrument.

Features

New analytes

- QuikRead go[®] HbA1c

Improvements

- Possibility to disable emergency button from login screen is added
- Password can be reset even when login is not used
- GPrinter 5890 XIII paper roll printer support added
- Company name change

Availability of new software

- After release the new software version 8.1.1 is available on a per customer basis
- After QuikRead go[®] HbA1c test launch the new software version 8.1.1 is downloadable from <http://www.softwareupdate.quikread.com/>
- After QuikRead go[®] HbA1c test launch the new 8.1.1 software can be also be ordered on a USB memory stick, cat.no 137540 QuikRead go Software USB stick