




Acro 2019-nCoV IgG/IgM Rapid Test

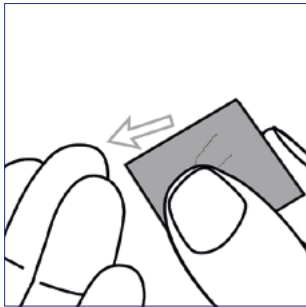


1 Remove the cassette from the foil pouch and place it on a clean and level surface. Take the sample from the side of the middle finger or the ring finger. Use alcohol pad to clean the fingertip.

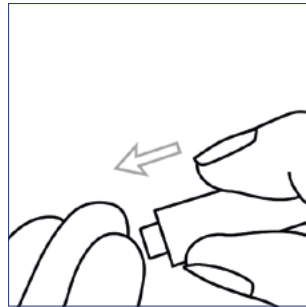


2 Place the lancet firmly against the skin and puncture the skin. Squeeze and wipe off the first blood drop.

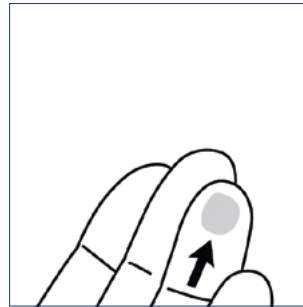


3 To increase the blood flow, squeeze the finger firmly but not continuously.

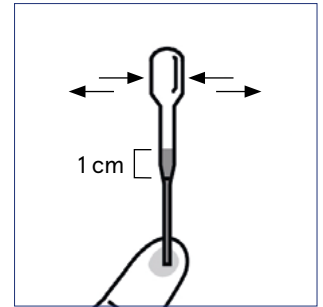


4 Squeeze the pipette bulb. Insert the tip to the blood drop. Release the pressure slowly to avoid taking in air. Fill the pipette up to 1 cm above the fill line.

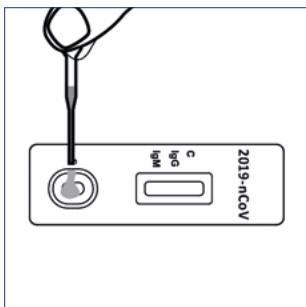

5 Transfer 1 full drop of blood (approximately 20 μ l) to the sample well (S).

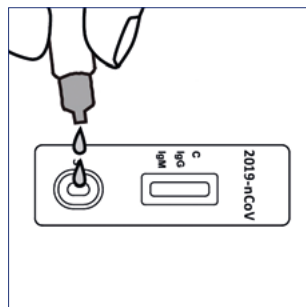


6 Add 2 drops of buffer (approximately 80 μ l) and start the timer.

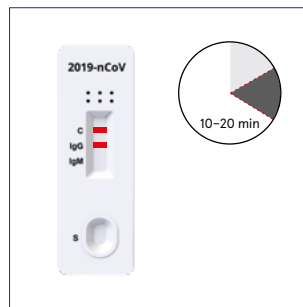

7 Read the results at 10 minutes. Do not interpret the results after 20 minutes.


Please consult the instructions for use before performing the test.

Acro 2019-nCoV IgG/IgM Rapid Test

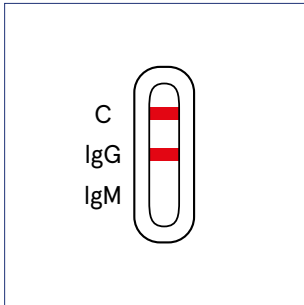


IgG positive result

Two coloured lines appear. One coloured line appears in the control line region (C) and another line appears in the IgG region.

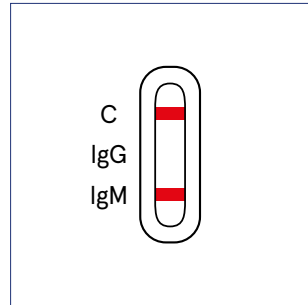


IgM positive result

Two coloured lines appear. One coloured line appears in the control line region (C) and another line appears in the IgM region.

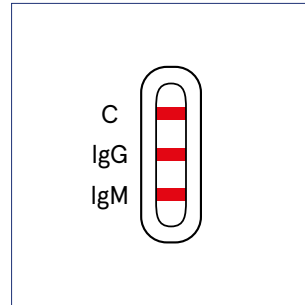


IgG and IgM positive result

Three coloured lines appear. One coloured line appears in the control line region (C), one line appears in the IgG region and one line appears in the IgM region.

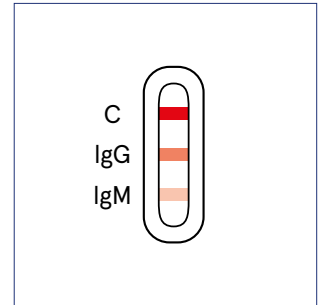


NOTE: The intensity of the colour may vary depending on the concentration of 2019-nCoV antibodies present in the sample. Any shade of color in the test line should be considered positive.

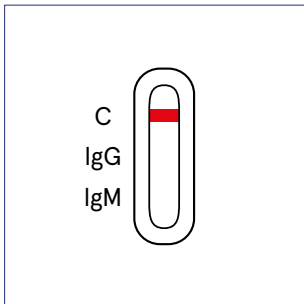

Negative result

One coloured line appears in the control line region (C). No line appears in the IgG or IgM region.

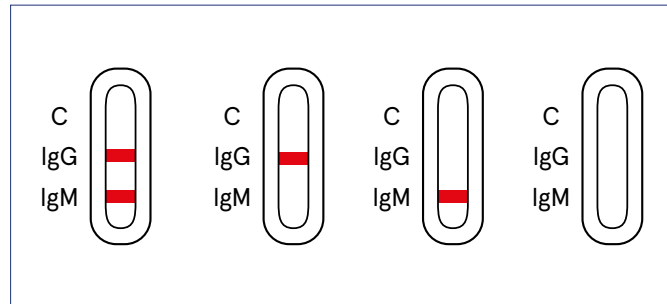

Invalid result

Control line (C) fails to appear. Review the procedure and repeat the test with a new test. If the problem persists, discontinue using the test kit and contact Aidian's representative.

For professional *in vitro* diagnostic use only.

Contact info

Aidian Oy

aidian@aidian.eu

+358 10 309 3000

For orders: orders@aidian.eu

www.aidian.eu

Product	Product no.	Order no.
Acro 2019-nCoV IgG/IgM Rapid Test Cassette, 25 tests	INCP-402	154601